



Stop face shield and goggle fogging with these five steps:


Instructions

Steps for applying the products to eye protection for anti-fogging:



Step 1

Wipe down eye protection with a clean microfibre cloth to make sure there is no dust.

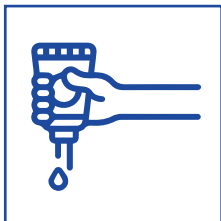

Step 2

Rub dish soap onto each side thoroughly for about a minute using fingers or microfibre cloth.

Face Shield use a nickel-sized amount ($\frac{1}{2}$ tsp), & a dime-sized ($\frac{1}{4}$ tsp) for **Goggles**



Step 3

Let the product sit on the eye protection for about a minute.



Step 4

Rinse the eye protection with warm water until it is clear.



Step 5

Air dry eye protection or wipe lightly with a clean, fine microfibre cloth.

With proper disinfecting between uses, this application should let you see clearly for 2-4 client visits.